



Training Day

2020

Agenda

Choosing the right Seating solution

How the Special Tomato Range meets your clients needs!

Any questions



Created by Tim Bergeron,
former Tumble Forms
industrial designer.

Tim sort to create a softer
material, more durable and
aesthetically pleasing to
parents.

A material able to be used
for long periods of time,
latex-free and built-in Anti-
microbial protection.



fiveyear
warranty



Why Special Tomato!



Soft-Touch® Material

The Soft-Touch® material outer skin prevents the products from peeling or tearing and also prevents liquids from penetrating the core. It is cool and smooth reducing the possibility of children's skin sticking to the seating surface. The soft-to-the-touch inner core is a specially formulated foam that allows for children to sit comfortably for longer periods of time.



Safety Concern

With over 35 years of manufacturing pediatric seating and positioning systems, our products are safe and easy to use while providing a latex-free contact surface. The contoured designs of our products provide a large surface area for distributing pressure which reduces the risk of skin shear and breakdown. Many of our products are crash-tested for auto and FAA approved for airline use.



Design and Function

Hand-crafted in upstate New York, the Special Tomato® products are designed with the needs of the user and care provider in mind. The superior Soft-Touch® materials are very durable and easy to clean. Our unique bent wooden bases deliver support and strength while the Soft-Touch® material allows our designers to subtly integrate the positioning contours needed for a proper seating system. We stand behind the quality of our products by offering up to a 5 year warranty on most of our products.



Hi-Lo
MPS Seating
Soft Touch Sitters
Soft Touch Liners
Potty Seat
Therapy rolls
RSS



EIO stroller
Jogger Pram
Activity Tray



Choosing the right solution

With so many solutions available, how can we find the right seating solution:

Where will you use your seating solution?



Home: Special needs seating systems at home can be diverse. What role will the seating have in the home? What level of versatility is required?, what applications does it need to fulfill? Does it need to adjust the seat height to countertops, down on the floor, and everywhere in between.



School: In a school setting, does the seating system need to be mobile and have the ability to keep a child positioned in their seating system, but still move them around the classroom to different stations as well as move them around the school? This could cut down on the need for multiple seating systems saving the school money. It will also allow the child to spend more time learning.



Around the Town: Does your seating solution need to be portable? An option for children who need some support to sit up, but either don't require a wheelchair or, the wheelchair is too bulky to fit into a tight spot. Something light enough for the care givers to carry , versatile to use in a variety of situations.



Choosing the right solution

Will there be enough room for growth?

It is always important to order a special needs seating solution to fit your child now.

However, some seating solutions will provide years of growth for the user.

The most important measurements for fitting a special needs seating are:

Hip Width - corresponds with width of the seat

Chest Width - corresponds with width of the seat

Shoulder Width - corresponds with width of the seat

Seat to Top of Head - corresponds with height of the seat back

Seat to Top of Shoulders - corresponds with height of the seat back

Thigh Depth - corresponds to the seat depth

Foot to Back of Knee - corresponds to the footrest to seat height

Weight - corresponds with the maximum weight capacity of the stander






Choosing the right solution

What type of supports are required?

Special needs seating offer **extra support** to help the user maintain the correct sitting position.

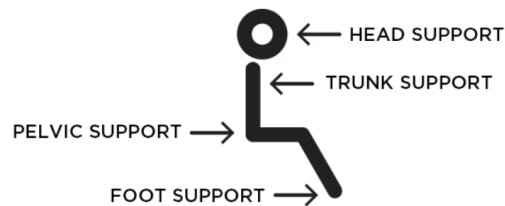

The amount of support offered will vary and it is important to know what areas of the body need assistance.

Some of the most common areas that require positioning accessories are: **head, trunk, pelvis and feet.**



A solution to meet most needs



NB: Sitter shown with Mobile base



Choosing the Right solution for Home



I need a versatile seating solution at home!

Various seating positions to be used throughout the home for different purposes.

Join the family at the table

Play with siblings at floor level.

Mobile to move around the home

Tilt to enable a period of rest

Adjustable table angle for casual feeding, reading, tablet, and play

Able to be used for long periods of time, latex-free and built-in Anti-microbial protection.



I need a chair to sit at
floor level and
elevated to table
height



Multi-Position System (MPS)



Available as a stand alone seat, or with Mobile Base or Hi-Lo base.

2 Shell sizes – Small (7-10") & Large (11 – 13")

Wide range of cushion sizes

Adjust the position of cushions as you grow.

Contoured soft touch cushion support.



-30° to 10° range of tilt-in-space.



30° of hip-flex adjustment.



Optional adjustable tray.



Floor to table height.

fiveyear
warranty





MPS Hi-Lo

	Small MPS Seat Cushions				Large MPS Seat Cushions			
MPS Seat Specifications	MS1	MS3	MS5	MS7	ML1	ML3	ML5	ML7
Target User Age	9 mos - 3 yrs	2 - 5 yrs	4 - 8 yrs	5 - 9 yrs	6 - 9 yrs	8 - 11 yrs	10 - 14 yrs	12 yrs - Adult
Target User Height	28" - 37"	32" - 43"	40" - 50"	43" - 52"	45" - 58"	48" - 60"	48" - 62"	58" - Adult
Maximum User Wt.	80 lbs.				130 lbs.			
Seat Width	7"	9"	10"	10"	11"	12"	13"	13"
Seat Depth	6"	8"	10"	12"	11"	13"	15"	17"
Seat to Footrest	6" - 16"				8" - 18"			
Hip Flex Angle	-15° to +15°				-15° to +15°			
Seat Tilt Angle Forward	5°				5°			
Seat Tilt Angle Back	-30°				-30°			
Floor to Seat Ht.	14" - 28.5"				17" - 28.5"			

	Small MPS Back Cushions		Large MPS Back Cushions	
MPS Back Specifications	MS2	MS4	ML2	ML4
Target User Age	9 mos - 3 yrs	2 - 9 yrs	6 - 9 yrs	8 yrs - Adult
Target User Height	28" - 37"	32" - 52"	45" - 58"	48" - Adult
Maximum User Wt.	80 lbs.		130 lbs.	
Back Width	8"	9"	10"	11"
Back Height	8.5"	11"	11"	13"
Top of Back Cushion	11" - 16"	15" - 20"	15" - 20"	19" - 22"
Top of Head Cushion	18" - 23"	21" - 26"	24" - 28"	26" - 30"

MPS Tray	Small (79060155)	Large (79060355)
Dimensions	24"W x 17"D x 2"H	
Weight	8 lbs.	

MPS Hi-Low Seating System
 79300109 Dark Gray, Small
 79300309 Dark Gray, Large
 Includes Seat Shell with Footrest,
 8-point Harness, Hi-Low Base with
 Casters (Must Choose Custom
 Seating Cushions Separately)



Standard Features Include:

Tilt-in-Space (with child in the seat) from -30 degrees to 10 degrees

Floor to Table Adjustment

Contoured Cushion Support with Soft-Touch® Material

Ergo 8-point Harness Design

30 degrees Hip-Flex® Adjustment for optimal pelvic positioning

Angle, Depth & Height Adjustable Footrest

Ergonomic Push Handle – comfortable grip for all heights

Pedal activated adjustments for Tilt-in-Space & Hi-Low functions

Four Swivel Locking Casters

Wood Accents for structure and a sleek, modern design

Tool Free Adjustments



Choosing the Right solution for Home



I need a supportive seat for feeding, for activities

Join the family at the table in the Special Tomato Sitter attached to the dining room chair

Play with siblings at floor level in a sitter on a mobile base or floor sitter

Mobile to move around the home

Adjustable Mobile Activity Tray table angle for casual feeding, reading, tablet, and play

Able to be used for long periods of time, latex-free and built-in Anti-microbial protection.



NB: Sitter shown with Mobile base





Recliner Support System



I need a supportive seat that integrates with my furniture

Adjustable padded platform for proper body alignment

Designed to safely adjust body position

Will work with most recliner brands

Easy to clean recliner drape and padded attachment straps
protect furniture

Includes: Small and Large 5-point Chest Harness,
3-point Pelvic Harness, Headrest & Leg Support

Need more support!

Compatible with Soft Touch Liners – Grow your seat width from 10" to 16"

New Hip Flex Seat Cushions With a "living hinge" to comfortably move with and support the body as it reclines.






Recliner Support System



- 5-Point Adjustable Chest Harness to maintain proper position of the trunk
- 3-Point Adjustable Pelvic Harness to maintain proper position of the pelvis and hips
- Both large & small size harnesses are included for growth!
- Height Adjustable Headrest to promote midline head position
- Leg Supports with height and lateral (side to side) adjustability to cradle lower extremities in neutral alignment and discourage hyper extension of knees
- Easy wipe clean and machine washable recliner drape
- Padded straps to secure RSS and protect furniture
- Perfect support for full recline while watching entertainment
- Completely portable and easy to store

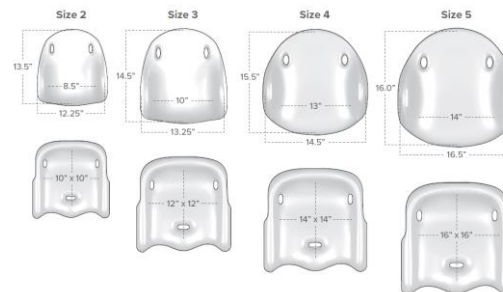

Special Tomato® Recliner Support System Size Chart	
Target User Height (approx.)	35" - 74" (89-188 cm)
Seat Width	20" (51 cm)
Seat Depth	18" (46 cm)
Seat Back Height	33" (84 cm)
Seat Back Height with Seat Liners	30" (76 cm)
Leg pad adjustment range	8" (20 cm)
Shoulder Harness Height Levels	14" - 27" (35 - 69 cm)
Seat Recline Range	Full range of furniture recliner
Overall Dimensions (width x length x height)	67" x 20" (170 x 51 cm)
Folded Dimensions (width x length x height)	36" x 21" x 10" (91 x 53 x 25 cm)
Maximum Weight Capacity	furniture dependent
Weight of RSS	10 lbs (4.5 kg)



Choosing the Right solution for School



I need a mobile supportive seat for every activity!

The ability to freely move from point A to point B is essential for both child and caregiver

Interacting & engaging with Teacher and fellow students

One solution to be used for learning, mealtime, playtime and beyond

A solution to support both Active sitting and periods of rest





Sitters

- 3 Colours
- 5 Sizes –up to Teenager / Adult
- Portable & easy to transport
- Attachment strap for chairs, take to the restaurant, classroom, add support to strollers and wheelchairs.
- Includes 8-point harness
- Built in Abductor
- Contoured head and lateral supports
- Most popular solution is Sitter + Mobile Base + Mobile Activity Tray

Base Styles
Three styles to fit your needs



Sitter with Stationary Base



Sitter with Mobile Base



Floor Sitter Kit for Sizes 1,2,3

Applications
Sitters can be used in a variety of ways



Sitter Only
5 sizes available for that "custom" fit



Size 1 Size 2 Size 3 Size 4 Size 5

Customize Your Cushion Colour
Three colour options available



Lilac



Aqua



Dark Grey





Mobile Activity Tray

Available in 2 Sizes

- Short – 16 to 21"
- Tall – 20 to 28"

Height and Angle adjustable

Utilise with Sitters, Wheelchairs and Walkers

A **Tray** provides a surface for activities like eating, drinking, games and crafts. **Casters** provide an easy way to move the child from room to room without having to change adaptive equipment.





Choosing the Right solution for Around Town



I need a portable supportive seat for every setting!

To enjoy everything without limitation.

A solution that is safe, comfortable and a familiar seat.

A booster seat to take to a friend's house, suitable for feeding and playtime.

Can attach to dining room chairs, restaurant & cinema chairs, no sacrifice in comfort and support.

A portable floor chair to take to therapy or to interact with other kids at a friend's place.

Low-to-the-ground perfect for all-day play and exploration.

Compact and easy to take with me on holidays





Seat & Back Liners

4 Colours

5 Sizes – New Size 4 & 5



- Portable & easy to transport
- Attachment strap for chairs, take to the restaurant, classroom, add support to strollers and wheelchairs.
- Seat cushion includes 3-point harness

New Hip Flex Seat Cushions



- With a “living hinge” to comfortably move with and support the body as it reclines.



fiveyear
warranty





Out & About Seat



Padded chest & pelvic harness hold your child securely in place allowing them to discover their world.

Optional Special Tomato seat & back liner cushions enhance lateral support and comfort.

Use in classrooms, restaurants, sporting events, movie theatres, relative's homes when visiting

Assists your child for participation in classroom & social settings.

Provides comfortable upper body support.

Backpack straps keep your hands-free making traveling a breeze.

Even folds with Special tomato seat & back liners attached.

Lightweight, portable and easy to clean.

Need more support?

Compatible with Soft Touch Liners – Grow your seat width from 10" to 14"





Out & About Seat



Special Tomato® OAS Dimensions	
Seat Width	15" (38.1 cm)
Seat Depth	13" (33 cm)
Seat Back Height	27" (68.5 cm)
Seat Back Height with Seat Liners	25" (63.5 cm)
Minimum Chair Back Height	14" (35.5 cm)
Seat Base to Top Of Head Support Max	25" (63.5 cm)
Shoulder Harness Height Levels	11.5", 14", 16.5", 19", 21.5" (29.2cm, 35.5 cm, 6.49cm, 7.4cm, 8.4)
Overall Dimensions (width x length)	16" x 49" (40.6cm x 124.4 cm)
Folded Dimensions (width x length x height)	16" x 27" x 6" (40.6 cm x 68.5cm x 15.2cm)
Maximum Weight Capacity	90 lbs (40.8 kg)
Weight of OAS	7 lbs (3.17kg)



*The Out & About Seat
accepts seat & back liners*

Key Information



- ❖ Piece of mind 2 Yr warranty - the Out & About , Recliner Support System
5 Year warranty on the MPS, Sitters & Liners
- ❖ Versatility A Special Tomato solution for Home, School and on the Go.
Seating compatible with Strollers, Joggers, attach to your chair, to a floor base.
- ❖ Go anywhere Take a supportive seating solution to your friends, to the movies, on holidays.



Home of brands

Seating



Standing



Walking



Recreation



BY CONNIEHANSEN



And more..

