

# Assembly Instructions



MP20401 - Raised 2IN Toilet Seat with Arms

MP20402 - Raised 4IN Toilet Seat with Arms

## Check you have all the parts:

|    |  |
|----|--|
| 1x | Seat   |
| 1x | Left Armrest   |
| 1x | Right Armrest (long side of the arm at the back of the seat) |
| 6x | Screws   |
| 6x | Curved washers   |

## Tools required:

|    |                      |
|----|----------------------|
| 1x | Phillips screwdriver |
|----|----------------------|

**TIP:** Have an assembly surface that allows for overhanging so you can place the armrest securely.

## Instructions

1. Place seat upside down on the assembly surface
2. Place armrest in groove on the seat with long side of frame at the back
3. Place curved washers (with wings hugging the armrest frame) and screw into the screw holes (tighten loosely)
4. Repeat for remaining 2 holes **TIP:** Start with the hole at the end of the long arm first
5. Repeat for opposite arm
6. Tighten all screws evenly
7. Completed Assembly

## Guidance for Use

To fasten to toilet seat, loosen the butterfly screws on the 3 mounts, and tighten to suit the toilet seat size

Maximum Weight Capacity: 190kg

