



Swing-away Joystick Mounting Bracket Adjustment Instructions

Tools Needed:

- 2.5 mm hex head key
- 6 mm hex head key

Turn off the power to the controller and ensure the power chair is not in freewheel mode. Refer to the freewheel operation label on the power chair.

To adjust home position, hold item A2 when tightening or loosening fastener item A1. After loosening A1, set center piece in desired position for home position. Then retighten A1, while continuing to hold A2.

To adjust controller offset, insert 2.5 mm hex head key into key hole underneath item A3. Hold the hex head key in this position, then loosen fastener A4 with 6 mm hex head key. Controller offset can then be adjusted to desired position. After desired position is achieved, continue to hold 2.5 mm hex head key in key hole, and retighten fastener A4 with 6 mm hex head key.

NOTE: DO NOT hold item A2 when tightening/loosening fastener A4. This may cause unintended tension on the internal belt and may damage internal components.

NOTE: The belt tension is preset. It should be adjusted only if the controller does not maintain its orientation when the controller swings left or right. To adjust belt tension, remove the label with review adjustment guide.

NOTE: DO NOT overtighten the belt tension or you may damage the swing-away module, and it will not work as intended. If adjustment is needed, rotate the setscrew only one-quarter turn at a time.

WARNING!

Make sure the fasteners are fully secure and the joystick is in the home position before using your power chair.

WARNING!

Make sure the joystick is properly secured to the armrest before using your power chair.

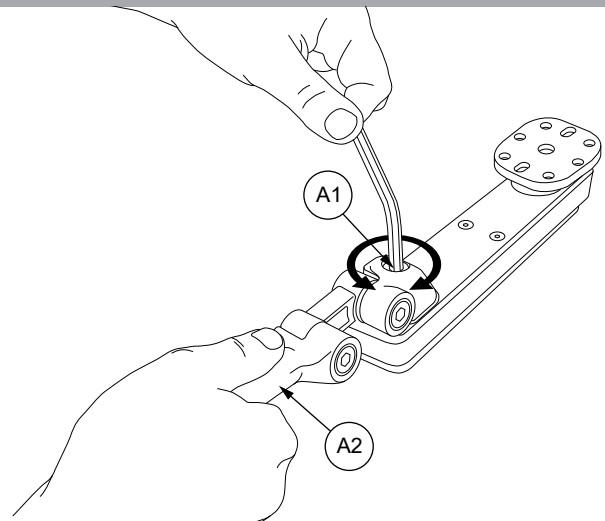


Figure 1.

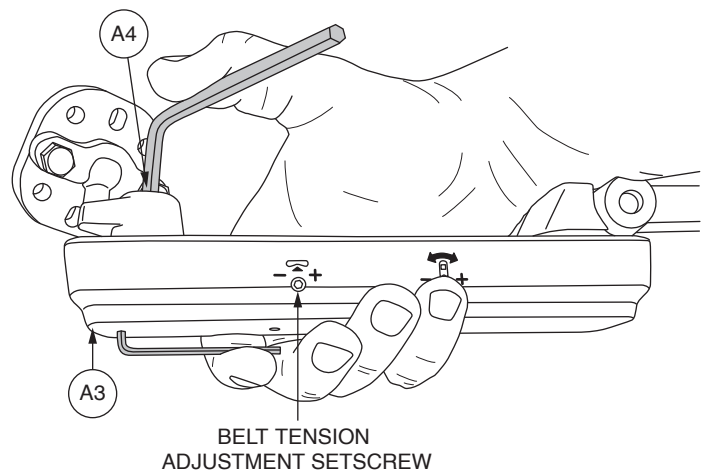


Figure 2.

